

How to care for your equipment

The proper maintenance of your equipment is necessary to provide the longest life possible.

Caring for your Gold Leads

We do not recommend using any abrasive materials when cleaning the 10-20 paste from the Gold Leads. This would include any type of facial tissue or paper towels. To receive the longest life out of your Gold Leads we recommend using only warm water. If you need to use them immediately after cleaning them blot them dry with a soft micro fiber cloth. If you do not need to use them immediately you can allow them to air dry.

Caring for your Neuro-Map Cap

The Neuro-Map Cap needs to be cleaned after every use. Clean the Neuro-Map Cap using warm water and a toothbrush. While running the sensors under warm water use the tooth brush to clean the Electro-Gel from inside the sensors. The fabric of the cap can be cleaned with ivory hand soap. It is recommended that you allow the cap to air dry after washing. If you need to use it immediately place a towel inside the cap to absorb the dampness.