



How to adjust band pass filters in Bio-Explorer


1. Click on "Tools" in the top toolbar in Bio-Explorer then click on filters.

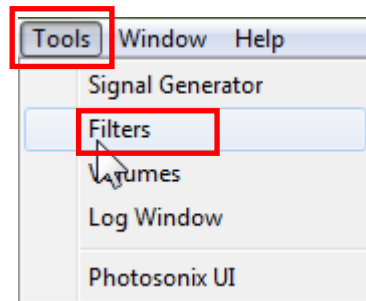

2. In the filters menu choose the frequencies you want to adjust and double click on it. The filters are set up as left down first, right down, left up, right up.

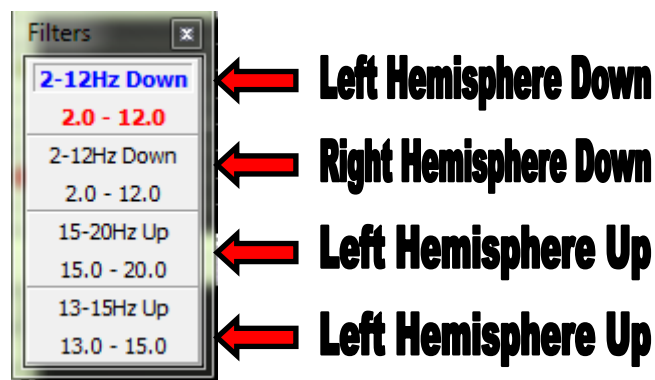

3. In the Band pass Filter properties menu adjust the low and high setting to reflect the frequencies you would like to change. Be sure to click on the objects tab and change the name to reflect the changes (see next page for example). When you have made your changes click on apply then ok.

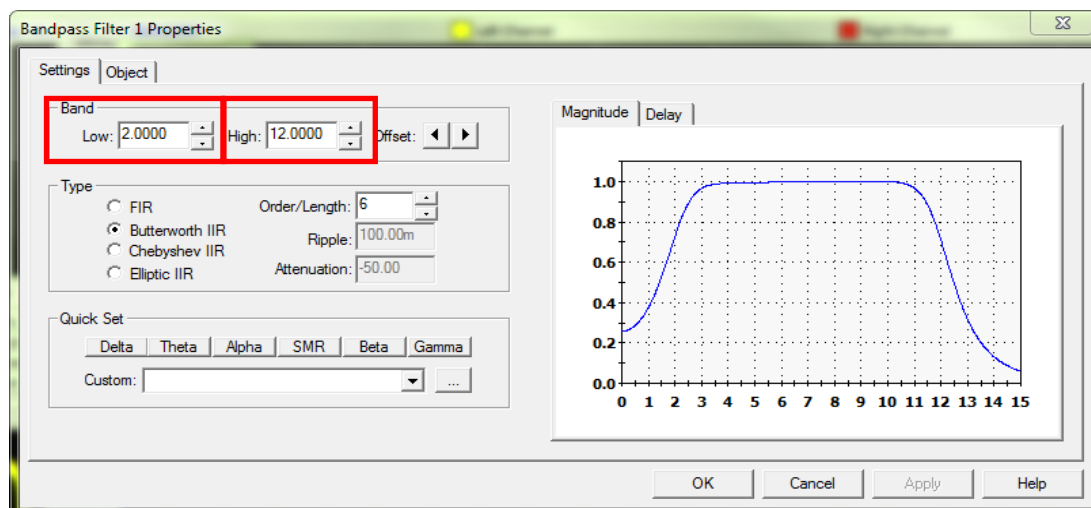

The name are below will need to be changed to reflect the new frequency changes. For example if you changed a filter from 15-20 Hz up to 13-15 Hz up you will need to enter the new frequencies of 113-15 Hz here.

