



Adjusting Video & Audio Codecs

Occasionally a movie will not play with the video or audio codecs that were chosen when your system was loaded by Clear Mind Center. We have loaded multiple video and audio codecs on your system to correct this issue. The following instructions will guide you through the process of changing these. Two problems that can occur are: “failed to create player” or the movie screen may jump around once the session has started. If either of these problems occur follow the steps below to change the video codec.


1. Left click on “BioExplorer” in the toolbar at the top left of the screen.

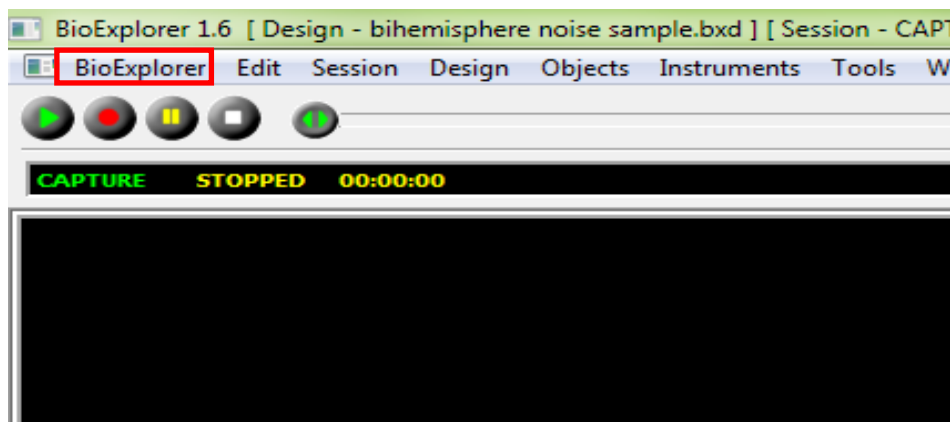



2. Left click on “Preferences” in the drop down menu.

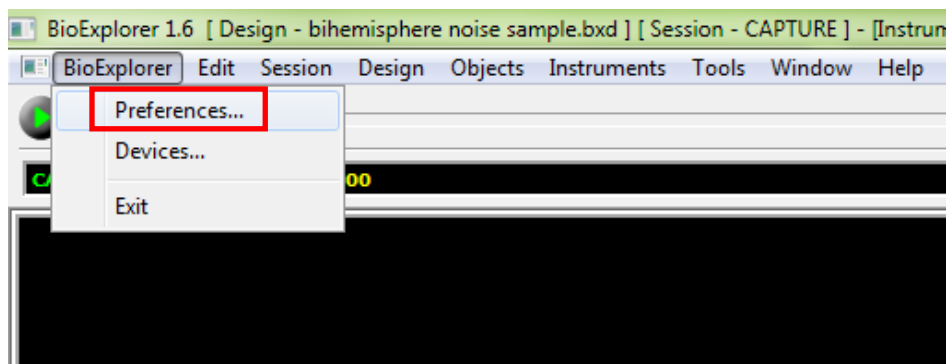

3. Left click on the “DVD Settings” tab in the Preferences Properties window.

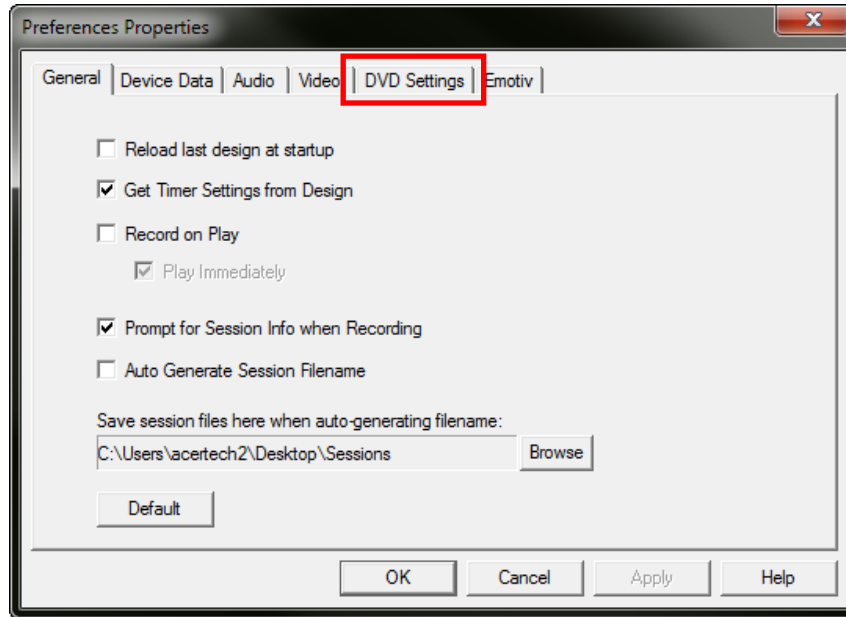

4. Left click on the down facing arrow to the right of the codec then choose the new codec from the drop down menu. Then left click on “Apply” then left click on “OK”.

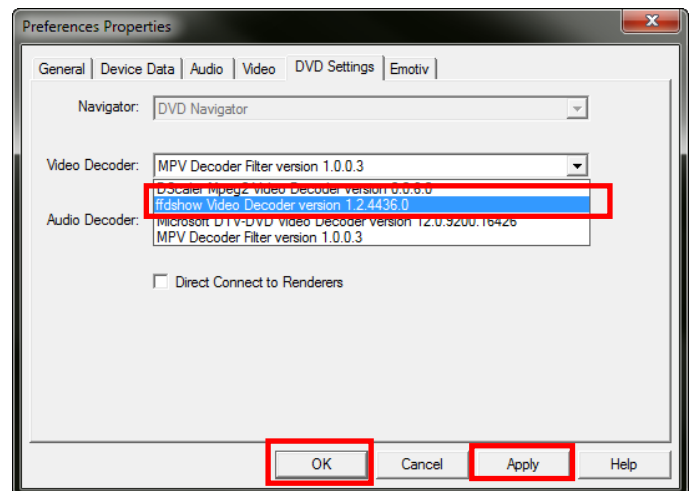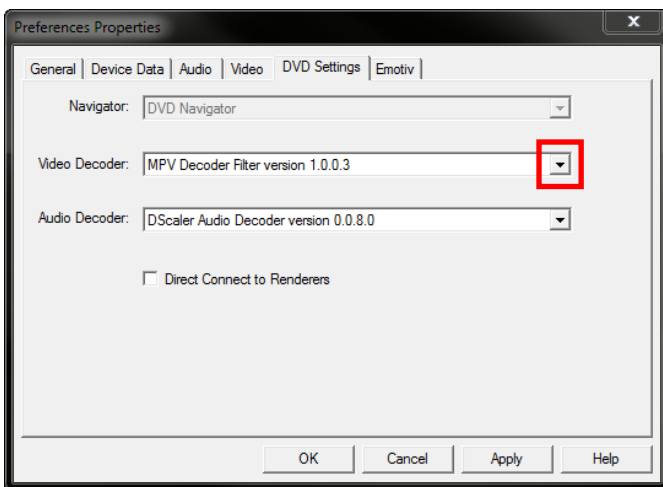

5. After left clicking on “OK” the properties window will close and you can now test your new codec.